

MAGic EyeMerge Hot Keys

The following hot keys use the **F12** modifier, which is the application's default modifier key. You must press the modifier key in combination with the hot key to activate the command. If you change the default modifier key, substitute the modifier key that you are using when issuing a command. Refer to the MAGic EyeMerge help for more information about the modifier key.

Zoom and Focus

| Description | Modifier Key+Hot Key |
|---|----------------------|
| Increase zoom | F12+F2 |
| Decrease zoom | F12+F1 |
| Find – zoom out with crosshairs; press again to remove crosshairs and return to previous zoom level | F12+A |
| Focus lock – turn Auto Focus on or off (lock) | F12+D |
| Freeze Frame toggle | F12+Z |

Screen View

| Screen View Description | Modifier Key+Hot Key |
|--|---|
| Select one of three views: Full Screen – Camera view, Full Screen – Computer view, or Split Screen view | F12+V (add SHIFT to reverse order) |
| Full Screen – Camera view | F12+HOME |
| Full Screen – Computer view | F12+END |
| Split Screen view | F12+PAGE UP |
| Split Screen orientation – toggle Split Screen view between horizontal and vertical orientation; use this keystroke in conjunction with F12+F to change camera view locations | F12+P |
| Move Split Screen view – toggle the camera position to move the Split Screen view from top to bottom or from left to right based on screen orientation (F12+P) | F12+F |

| Screen View Description | Modifier Key+Hot Key |
|---|---|
| Increase Split Screen viewing area | F12+F5 |
| Decrease Split Screen viewing area | F12+F6 |
| Camera flip – switch between Normal, Mirror, Upside Down, or Mirror Upside Down views | F12+Q (add SHIFT to reverse order) |
| Move EyeMerge console to foreground | F12+ENTER |
| Scroll image up or left while in Split Screen mode | F12+F8 |
| Scroll image down or right while in Split Screen mode | F12+F7 |

Video Modes

| Video Modes Description | Modifier Key+Hot Key |
|---|---|
| Select Color Video mode – switch between Normal color mode, Negative (inverted color) mode, Black on White (positive) mode, or White on Black (negative) mode | F12+E (add SHIFT to reverse order) |
| Normal (full color) mode | F12+SHIFT+A |
| Negative (inverted color) mode | F12+SHIFT+S |
| Black on white (positive) mode | F12+SHIFT+D |
| White on black (negative) mode | F12+SHIFT+F |

Capture Image

| Capture Image Description | Modifier Key+Hot Key |
|---------------------------|----------------------|
| Save current screen image | F12+CTRL+S |
| Capture image | F12+TAB |
| Delete image | F12+DELETE |

View Image

| View Image Description | Modifier Key+Hot Key |
|------------------------|---------------------------|
| Jump to live image | F12+0 (zero) |
| Next image | F12+SPACEBAR |
| Previous image | F12+SHIFT+SPACEBAR |
| Jump to image 1 | F12+1 |
| Jump to image 2 | F12+2 |

| View Image Description | Modifier Key+Hot Key |
|-------------------------------|-----------------------------|
| Jump to image 3 | F12+3 |
| Jump to image 4 | F12+4 |
| Jump to image 5 | F12+5 |
| Jump to image 6 | F12+6 |
| Jump to image 7 | F12+7 |
| Jump to image 8 | F12+8 |
| Jump to image 9 | F12+9 |
| Open image file manager | F12+G |

Lines and Masks

| Lines and Masks Description | Modifier Key+Hot Key |
|---|-----------------------------|
| Screen markers – toggle on or off | F12+K |
| Screen marker style – select reading lines or screen masking | F12+Y |
| Screen marker orientation – toggle between horizontal or vertical screen marker orientation | F12+O |
| Move reading lines or screen mask down or to the right | F12+N |
| Move reading lines or screen mask up or to the left | F12+SHIFT+N |
| Increase distance between reading lines or increase amount of image viewable in the masked area | F12+F10 |

| Lines and Masks Description | Modifier Key+Hot Key |
|---|---|
| Decrease distance between reading lines or decrease amount of image viewable in the masked area | F12+F9 |
| Select reading line color | F12+R (add SHIFT to reverse order) |

Camera Control

| Camera Control Description | Modifier Key+Hot Key |
|----------------------------|----------------------|
| Camera power on or off | F12+X |
| Refresh video view | F12+ESCAPE |
| Change the camera source | F12+S |

Picture Adjustment

| Picture Adjustment Description | Modifier Key+Hot Key |
|--------------------------------|----------------------|
| Increase brightness | F12+B |
| Decrease brightness | F12+SHIFT+B |
| Increase hue | F12+H |
| Decrease hue | F12+SHIFT+H |
| Increase saturation | F12+U |
| Decrease saturation | F12+SHIFT+U |
| Increase contrast | F12+C |
| Decrease contrast | F12+SHIFT+C |