








TOPAZ Control Panel

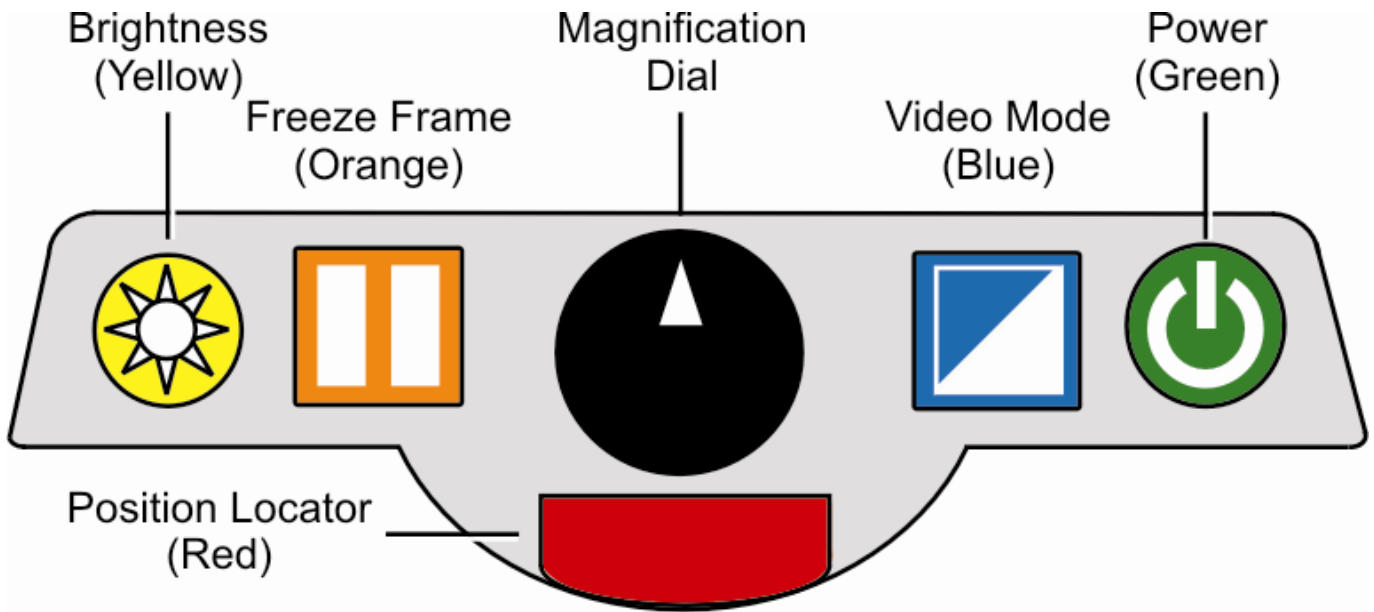

Dial or Button	Location	Description
	The Brightness dial is located at the far left on the control panel.	Turn this dial to increase or decrease brightness. Use it to turn the lights on and off by pressing the Brightness dial and the Freeze Frame button simultaneously.
	The Freeze Frame button is the second button from the left on the control panel.	Press this button to freeze the current image on the screen. Press it again to turn off Freeze Frame mode. Use it to turn the lights on and off by pressing the Brightness dial and the Freeze Frame button simultaneously.
	The Magnification dial is located in the center of the control panel.	Turn this dial to increase or decrease magnification.

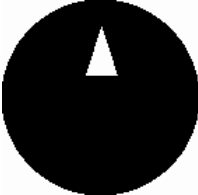Dial or Button	Location	Description
	<p>The Position Locator button is located directly underneath the Magnification dial in the center of the control panel.</p>	<p>Press this button to turn the Position Locator beam on or off.</p>
	<p>The Video Mode button is the second button from the right on the control panel.</p>	<p>Press this button to cycle through the available video modes.</p>
	<p>The Power button is located at the far right on the control panel.</p>	<p>Press this button to turn the power on for the monitor, camera, and lights. Press and hold this button to turn the power off.</p>

Should you need assistance please contact our technical support department at 727-803-8600.

You will need your serial number in order for our support department to assist you:

Serial number: _____

Date of purchase: _____

Sales Contact: _____
